



» BIPOLAR FORCEPS «





Tekno-Medical Optik-Chirurgie GmbH

Sattlerstr. 11

78532 Tuttlingen

GERMANY

SRN: DE-MF-000005822

Phone: +49 (0) 7461 / 17 01 0

Mail: mail@tekno-medical.com

Web: www.tekno-medical.com



Content

1	SCOPE	4
2	INSPECTIONS	4
3	HANDLING	4
4	PURPOSES	4
5	INDICATION	4
6	CONTRAINDICATIONS	4
7	PATIENT POPULATION	4
8	DISPOSAL	4
9	WARNINGS	5
9.1	GENERAL SAFETY-NOTES	5
9.2	HF-SPECIFIC SAFETY-NOTES	5
10	COMBINATIONS	5
10.1	LENGTH OF THE ACCESSORIES	5
10.2	RF CABLE	5
11	REPROCESSING	6
11.1	ON-SITE PREPARATION	6
11.2	TRANSPORT	6
11.3	PREPARATION FOR DECONTAMINATION	6
11.4	MANUAL PRE-CLEANING	6
11.5	AUTOMATED CLEANING	6
11.6	AUTOMATED (THERMAL) DISINFECTION	7
11.7	FUNCTIONAL TESTING	7
11.8	PACKAGING	7
11.9	STERILIZATION	7
11.10	STORAGE	7
11.11	INFORMATION ON THE VALIDATION OF THE REPROCESSING	7
12	ADDITIONAL INSTRUCTIONS	7
13	REPORTING PRODUCT PROBLEMS	8
14	WARRANTY	8
15	SERVICE AND REPAIR	8
16	SYMBOLS	8
17	PRODUCT LISTING	9



In order to keep risks to patients, users or third parties as low as possible, the instructions for use must be carefully followed. The use, reprocessing and testing of the instruments may only be carried out by trained specialists. Before using the instrument, the entire instructions for use must be read. This also applies to the instructions for use of the accessories used.



The re-usable bipolar forceps from Tekno-Medical Optik-Chirurgie GmbH (Tekno) and their accessories are supplied non-sterile and must undergo the complete reprocessing cycle (cleaning, disinfection and sterilisation) before the first and each subsequent use.

1 SCOPE



These instructions for use are valid for the **bipolar forceps** (hereinafter referred to as "**forceps**") from Tekno-Medical Optik-Chirurgie GmbH. (See article list in the last paragraph of these instructions for use).

2 INSPECTIONS

Before each use of the forceps, they must be checked for breaks, cracks, deformation, damage and functionality. Areas such as insulation, connections and working ends must be checked particularly carefully. Worn, corroded, deformed, porous or otherwise damaged instruments must be discarded.

3 HANDLING

The products may only be used for their intended purpose by appropriately trained and qualified personnel. The treating physician or user is responsible for the selection of the instruments for specific applications or surgical use, the appropriate training of the staff and the experience in handling the products. All surgical instruments should always be handled with the utmost care when transporting, cleaning, maintaining, sterilizing and storing. This particularly applies to fine tips and other sensitive areas.

4 PURPOSES

Bipolar forceps are used to grasp, prepare and coagulate tissue. They must be connected to the bipolar output of an HF generator using a suitable bipolar cable and may only be operated with bipolar coagulation current. Activation is usually done using a foot switch.

Maximum operating voltage of the generator: 500 Vp!

5 INDICATION

The bipolar forceps from Tekno-Medical Optik Chirurgie GmbH are generally used in all areas of open surgery.

6 CONTRAINDICATIONS

The instrument is not intended for use on the central nervous and circulatory system.

Risks from improper use:

- Material fatigue and loss of function due to exceeding the product service life.
- Risk of electric shock due to damage to the insulation, which can occur if the operating conditions are exceeded and the reprocessing instructions are disregarded.

7 PATIENT POPULATION

Apart from the contraindicated uses listed in these instructions for use, there are no restrictions on the patient population.

8 DISPOSAL

If the instruments can no longer be repaired and reconditioned, the instruments must be disposed of in accordance with the applicable country-specific regulations and laws.



9 WARNINGS



Failure to observe these application and safety instructions can lead to injuries, malfunctions or other unexpected incidents! When using bipolar tweezers, the active electrode and the neutral electrode must be 100% surrounded by conductive rinsing fluid (0.9 percent saline or Ringer's solution); never use non-conductive rinsing fluids.

9.1 General safety-notes

- Do not grasp the distal end.
- Do not use or repair damaged instruments.
- Do not bend the distal end.
- The transport packaging is unsuitable for the high temperatures during autoclaving and must be discarded before the first sterilisation.
- Do not overload the instruments. Overloading due to excessive force can lead to breakage, bending and malfunction of the medical device and to injury to the patient or user.
- Do not bend bent instruments back into their original position, risk of breakage.

9.2 HF-specific safety-notes

- Risk of burns due to HF current
- The instrument may only be used by qualified, medically and technically trained personnel.
- For patients with pacemakers, check their compatibility with HF radiation.
- Do not use explosive / flammable substances during the operation.
- Do not place the instrument on the patient.
- Avoid carbonisation of the tissue!
- The power of the HF generator must always be set as low as possible in order to achieve only the desired effect.
- Do not use the instrument for spray coagulation.
- Instruments that are temporarily not in use must always be isolated from the patient in order to avoid patient injury in the event of accidental activation of the HF current.
- Only activate the HF current if the contact surfaces are within the visual range and have good contact with the tissue to be treated. Do not touch any other metallic instruments, trocar sleeves, optics, cables or similar.
- Remove disinfectant residues from the patient's body.
- Only use the instrument if the insulation is undamaged.
- Only touch the insulated areas with your fingers, not the contact pin.
- Always position the patient leads so that there is no contact with the patient or other leads.

Always check the forceps for:

- visibly exposed metal of the shaft at the connection point to the cable,
- poor electrical connection between the forceps and the cable,
- poor fit between the forceps and the cable.

To connect and disconnect the cable, always hold the plug only, never pull on the cable. The use of damaged cables can lead to considerable danger. Check the cable for visible damage before each use.

Damaged HF cables must not be used!

10 COMBINATIONS

10.1 Length of the accessories

Note regarding accessories that can be considered antennas (according to DIN EN IEC 60601-2-2, subsection 202.7.9.2.14 k):

**The length of the tweezers ranges from 30 to 170 mm.
The cable length is between 3 and 5 meters.**

10.2 RF cable

The forceps are connected to RF generators via bipolar RF cables.

**11 REPROCESSING**

In general, surgical instruments may only be reprocessed by persons who have the necessary expertise for the intended activities. Detailed information on the reprocessing of surgical instruments can be found in the "**Red Brochure**" of the AKI. Under www.a-k-i.org you will also find links to laws, standards and publications of processing expert committees.

Due to the product design and the materials used, no defined limit of maximum feasible applications can be set. The service life of medical devices is determined by their function and gentle handling. Frequent reprocessing has little effect on the product. The end of product life is usually determined by wear and damage caused by use. The legibility of the marking has been verified over 200 preparations.

11.1 On-site preparation

Immediately after use, remove coarse dirt from the instruments. Do not use any fixing agents or hot water (>40°C), as this will cause residues to freeze and may affect the success of cleaning.

11.2 Transport

Safe storage in a closed container and transport of the instruments to the reprocessing site to avoid damage to the instruments and contamination to the environment.

11.3 Preparation for decontamination

If possible, the instruments must be disassembled or opened for reprocessing. The instruments must be stored on machine-compatible instrument carriers in a dishwasher-safe manner. The condition of the instrument panels must not impair the subsequent cleaning and disinfection by means of sound or rinsing shadows.

11.4 Manual pre-cleaning

For manual cleaning / pre-cleaning, never use metal brushes, metal sponges or abrasive cleaning agents. Place the instruments in cold water for at least 5 minutes. If possible, disassemble the instruments and clean them under cold water with a soft brush until no residue is visible. Place instruments in an ultrasonic bath at 40°C for 15 minutes with 0.5% alkaline or enzymatic cleaner and sonicate. Remove instruments and rinse with cold water. The cleaning solution should be changed at least once a day, more often if necessary. Too much contamination impairs the cleaning effect and increases the risk of corrosion. National laws and guidelines must be observed.

11.5 Automated cleaning

Place the opened instruments in a sieve tray on the slide-in carriage and start the cleaning process. Disassemble instruments as much as possible.

Step	Parameter	
Pre-rinsing 1	Rinsing-temperature + water quality	Cold tap water
	Exposure time	60 s
Pre-rinsing 2	Rinsing-temperature + water quality	Cold tap water
	Exposure time	180 s
Cleaning	Cleaning-temperature	45 °C
	Water quality	Tap water
	Exposure time	300 s (worst case condition) / RKI Recommendation: 600 s
	Detergent	Neodisher Medizym
	Concentration	0,50 %
Neutralization	Rinsing-temperature	40 °C
	Water quality	Tap water
	Exposure time	180 s
	Neutralizing detergent	Neodisher Z
	Concentration	0,10 %
Post-rinsing	Rinsing-temperature	40 °C
	Water quality	Deionized water
	Exposure time	120 s

**11.6 Automated (thermal) disinfection**

Step	Parameter	
Thermal Disinfection	Disinfection-temperature	90 °C (A ₀ 3000)
	Water quality	Deionized water
	Exposure time	300 s
Drying	Drying of the outside of the instruments by the drying cycle of the washer-disinfector. If necessary, manual drying can also be achieved with the help of a lint-free cloth.	

For UK: The washer disinfection cycle should run at a minimum of 90 °C for a minimum of 1 minute.

11.7 Functional testing

After each cleaning, the products must be macroscopically clean, i.e., free of visible contamination. Stained products must be sorted out immediately and given special treatment. If errors or damage occur, the products must be sorted out immediately. Functional testing and maintenance of the instruments must be carried out extremely thoroughly. A suitable maintenance procedure increases the service life of the instruments.

11.8 Packaging

Select standard-compliant packaging of the instruments for sterilization according to DIN EN ISO 11607-1, DIN EN 868-2 and DIN EN 868-8.

11.9 Sterilization

Sterilization of the products using a fractionated vacuum process (according to DIN EN ISO 17665), taking into account the respective national requirements.

Pre-vacuum	3 times
Sterilization temperature	134 °C
Sterilization time	5 min
Drying time	20 min.

The use of other sterilization methods is beyond our responsibility.

Sterilization parameters UK:

Sterilization of the instruments by applying a fractionated pre-vacuum process (according to Health Technical Memorandum 01-01 Part C):

- Heat up to a minimum sterilization temperature of 134° - 137 °C (maximum temperature 137 °C).
- Minimum holding time: at least 3 min.

11.10 Storage

The sterilized instruments must be stored in suitable packaging in a dry, clean and dust-free environment and at a constant level of humidity. The distance between the floor and the shelf should be at least 30cm.

The storage period is to be determined by the user himself.

11.11 Information on the validation of the reprocessing

The following materials and machines were used in the processing process:

Detergent	Neodisher Medizym 0,5 % (v/h)
Neutralisator	Neodisher Z 0.1% (v/v)
Washer-disinfector (RDG)	Miele PG 8535
Steam-autoclave	Lautenschläger ZentraCert

For details see test reports: 23277 / 23278 / 23279 (CleanControlling Medical GmbH & Co. KG, 08-2021)

12 ADDITIONAL INSTRUCTIONS

If the chemicals and machines described above are not available, it is up to the user to validate his process accordingly. It is the responsibility of the user to ensure that the remanufacturing process, including resources, materials and personnel, is suitable to achieve the required results.

The state of the art and national laws require the following of validated processes.

During reprocessing, the temperature acting on the instrument should **not** exceed **140 °C**.

In principle, mechanical cleaning and disinfection are always preferable to manual cleaning. With mechanical cleaning and disinfection, there is greater safety in the process.

Never use metal brushes, metal sponges or abrasive cleaning agents for manual cleaning/pre-cleaning. Strongly alkaline cleaning agents damage plastics and anodized coatings. Do not use caustic cleaning agents. Do not use strong oxidizing cleaning agents. Agents with a neutral pH value (7.0) are best suited.



13 REPORTING PRODUCT PROBLEMS



In accordance with the requirements of Regulation (EU) 2017/745 on medical devices and our quality management system, all product problems must be reported to the manufacturer.

business hours you can reach us by phone at +49 (0) 7461 / 1701-0.

Outside of regular business hours, please send an email to safety@tekno-medical.com.

Serious incidents must also be reported to the local authority responsible for their location.

14 WARRANTY

The products are manufactured from high-quality materials and undergo quality control before delivery. Should any defects occur, please contact our service department. Tekno-Medical cannot guarantee that the products are suitable for any specific procedure. Tekno-Medical accepts no liability for accidental or consequential damages. Tekno-Medical accepts no liability if these instructions for use have been demonstrably violated.



Caution : In the event of use of the instruments on patients with Creutzfeldt-Jakob disease or its variants (vCJD, BSE, TSE), Tekno-Medical disclaims all responsibility for reuse.

15 SERVICE AND REPAIR

Do not attempt any repairs or modifications to the product yourself. This is the sole responsibility and intended use of authorized manufacturer personnel.

Defective products must have undergone the entire refurbishment process before being returned for repair.

For returns, please use our RMA application form and decontamination certificate.

Forms available at: <https://www.tekno-medical.com/de/service/reparaturservice/>

16 SYMBOLS

The symbols used in this instruction and on the label have the following meaning according to EN ISO 15223-1:

	Attention!		Manufacturer
	Medical device		Date of manufacturing
	Non-sterile		Read the instructions for use
	Catalogue number		Protect from sunlight
	Lot-number		Store in a dry place
	Unique device identification		
	CE mark with the number of the notified body: mdc – medical device certification GmbH Kriegerstrasse 6, D – 70191 Stuttgart		



17 PRODUCT LISTING

Printed on 10.06.2026

90051-11	90101-20°	90151-16	90185-16N°	90190-22N	90194-20	90285-16°	90290-22N°
90051-11°	90101-20N	90151-16°	90185-18	90190-22N°	90194-20°	90285-16N	90290-24
90051-11N	90101-20N°	90151-16N	90185-18°	90190-24	90194-20N	90285-16N°	90290-24°
90051-11N°	90101-22	90151-16N°	90185-18N	90190-24°	90194-20N°	90285-18	90290-24N
90051-12	90101-22°	90151-18	90185-18N°	90190-24N	90194-22	90285-18°	90290-24N°
90051-12°	90101-22N	90151-18°	90185-20	90190-24N°	90194-22°	90285-18N	90291-16
90051-12N	90101-22N°	90151-18N	90185-20°	90191-15	90194-22N	90285-18N°	90291-16°
90051-12N°	90101-24	90151-18N°	90185-20N	90191-15°	90194-22N°	90285-20	90291-16N
90052-11	90101-24°	90151-20	90185-20N°	90191-15N	90194-24	90285-20°	90291-16N°
90052-11°	90101-24N	90151-20°	90185-22	90191-15N°	90194-24°	90285-20N	90291-18
90052-11N	90101-24N°	90151-20N	90185-22°	90191-16	90194-24N	90285-20N°	90291-18°
90052-11N°	90101-30	90151-20N°	90185-22N	90191-16°	90281-18	90285-22	90291-18N
90052-12	90101-30°	90151-22	90185-22N°	90191-16N	90281-18°	90285-22°	90291-18N°
90052-12°	90101-30N	90151-22°	90185-24	90191-16N°	90281-18N	90285-22N	90291-20
90052-12N	90101-30N°	90151-22N	90185-24°	90191-18	90281-18N°	90285-22N°	90291-20°
90052-12N°	90102-15	90151-22N°	90185-24N	90191-18°	90281-20	90285-24	90291-20N
90054-11	90102-15°	90151-24	90185-24N°	90191-18N	90281-20°	90285-24°	90291-20N°
90054-11°	90102-15N	90151-24°	90186-18	90191-18N°	90281-20N	90285-24N	90291-22
90054-11N*	90102-15N°	90151-24N	90186-18°	90191-20	90281-20N°	90285-24N°	90291-22°
90054-11N°	90102-18	90151-24N°	90186-18N	90191-20°	90281-22	90286-18	90291-22N
90054-12	90102-18°	90152-15	90186-18N°	90191-20N	90281-22°	90286-18°	90291-22N°
90054-12°	90102-18N	90152-15N	90186-20	90191-20N°	90281-22N	90286-18N	90291-24
90054-12N*	90102-18N°	90152-15N°	90186-20°	90191-22	90281-22N°	90286-18N°	90291-24°
90054-12N°	90102-20	90152-18	90186-20N	90191-22°	90281-24	90286-20	90291-24N
90055-11	90102-20°	90152-18°	90186-20N°	90191-22N	90281-24°	90286-20°	90291-24N°
90055-11°	90102-20N	90152-18N	90186-22	90191-22N°	90281-24N	90286-20N	90292-16
90055-11N*	90102-20N°	90152-18N°	90186-22°	90191-24	90281-24N°	90286-20N°	90292-16°
90055-11N°	90102-22	90152-20	90186-22N	90191-24°	90282-18	90286-22	90292-16N
90055-12	90102-22°	90152-20°	90186-22N°	90191-24N	90282-18°	90286-22°	90292-16N°
90055-12°	90102-22N	90152-20N	90186-24	90191-24N°	90282-18N	90286-22N	90292-18
90055-12N*	90102-22N°	90152-20N°	90186-24°	90191-30N°	90282-18N°	90286-22N°	90292-18°
90055-12N°	90102-24	90152-22	90186-24N	90192-15	90282-20	90286-24N	90292-18N
90057-11	90102-24°	90152-22°	90186-24N°	90192-15°	90282-20°	90286-24N°	90292-18N°
90057-12	90102-24N	90152-22N	90187-18	90192-15N	90282-20N	90287-18	90292-20°
90058-11	90103-06*	90152-22N°	90187-18°	90192-15N°	90282-20N°	90287-18°	90292-20N
90058-12	90103-15	90152-24	90187-18N	90192-18	90282-22	90287-18N	90292-20N°
90060-07	90103-15°	90152-24°	90187-18N°	90192-18°	90282-22°	90287-18N°	90292-22
90061-07	90103-15N	90152-24N	90187-20	90192-18N	90282-22N	90287-20	90292-22°
90062-07	90103-15N°	90153-15	90187-20°	90192-18N°	90282-22N°	90287-20°	90292-22N
90063-07	90103-18	90153-15N	90187-20N	90192-20	90282-24	90287-20N	90292-22N°
90067-07*	90103-18°	90153-15N°	90187-20N°	90192-20°	90282-24°	90287-20N°	90292-24
90100-15	90103-18N	90153-18	90187-22	90192-20N	90282-24N	90287-22	90292-24°



Instructions for use – Please read before use

10 / 10

90100-15°	90103-18N°	90153-18°	90187-22°	90192-20N°	90282-24N°	90287-22°	90292-24N
90100-15N	90103-20	90153-18N	90187-22N	90192-22	90283-18	90287-22N	90292-24N°
90100-15N°	90103-20°	90153-18N°	90187-22N°	90192-22°	90283-18°	90287-22N°	90293-16
90100-16	90103-20N	90153-20	90187-24	90192-22N	90283-18N	90287-24	90293-16°
90100-16°	90103-20N°	90153-20°	90187-24°	90192-22N°	90283-18N°	90287-24°	90293-16N
90100-16N	90103-22	90153-20N	90187-24N	90192-24	90283-20	90287-24N	90293-16N°
90100-18	90103-22°	90153-20N°	90187-24N°	90192-24°	90283-20°	90287-24N°	90293-18
90100-18°	90103-22N	90153-22	90188-20*	90192-24N	90283-20N	90289-16	90293-18°
90100-18N	90103-22N°	90153-22°	90189-16	90193-15	90283-20N°	90289-16°	90293-18N
90100-18N°	90103-24	90153-22N	90189-16°	90193-15N	90283-22	90289-16N	90293-18N°
90100-20	90103-24°	90153-22N°	90189-16N	90193-15N°	90283-22°	90289-16N°	90293-20
90100-20°	90103-24N	90153-24	90189-16N°	90193-16	90283-22N	90289-18	90293-20°
90100-20N	90104-20*	90153-24°	90189-18	90193-16°	90283-22N°	90289-18°	90293-20N
90100-20N°	90150-15	90183-20N°	90189-18°	90193-16N	90283-24	90289-18N	90293-20N°
90100-22	90150-15N	90184-16	90189-18N	90193-16N°	90283-24°	90289-18N°	90293-22
90100-22°	90150-15N°	90184-16°	90189-18N°	90193-18	90283-24N	90289-20	90293-22°
90100-22N	90150-16	90184-16N	90189-20	90193-18°	90283-24N°	90289-20°	90293-22N
90100-22N°	90150-16°	90184-16N°	90189-20°	90193-18N	90284-16	90289-20N	90293-22N°
90100-24	90150-16N	90184-18	90189-20N	90193-18N°	90284-16°	90289-20N°	90293-24
90100-24°	90150-16N°	90184-18°	90189-20N°	90193-20	90284-16N	90289-22	90293-24°
90100-24N	90150-18	90184-18N	90189-22	90193-20°	90284-16N°	90289-22°	90293-24N
90100-30	90150-18°	90184-18N°	90189-22°	90193-20N	90284-18	90289-22N	90293-24N°
90100-30°	90150-18N	90184-20	90189-22N	90193-20N°	90284-18°	90289-22N°	90776-17
90100-30N	90150-18N°	90184-20°	90189-22N°	90193-22	90284-18N	90289-24	90776-17°
90101-11N°	90150-20	90184-20N	90189-24	90193-22°	90284-18N°	90289-24°	90776-17N
90101-15	90150-20°	90184-20N°	90189-24°	90193-22N	90284-20	90289-24N	90777-17
90101-15°	90150-20N	90184-22	90189-24N	90193-24	90284-20°	90289-24N°	90777-17°
90101-15N	90150-20N°	90184-22°	90189-24N°	90193-24°	90284-20N	90290-18	90777-17N
90101-15N°	90150-22	90184-22N	90190-18	90193-24N	90284-20N°	90290-18°	90780-16
90101-16	90150-22°	90184-22N°	90190-18°	90193-24N°	90284-22	90290-18N	90780-19
90101-16°	90150-22N	90184-24	90190-18N	90193-30N°	90284-22°	90290-18N°	90782-20*
90101-16N	90150-22N°	90184-24°	90190-18N°	90194-15	90284-22N	90290-20	90782-22*
90101-16N°	90150-24	90184-24N	90190-20	90194-15N	90284-22N°	90290-20°	90800-00
90101-18	90150-24°	90184-24N°	90190-20°	90194-15N°	90284-24	90290-20N	90800-01
90101-18°	90150-24N	90185-07*	90190-20N	90194-18	90284-24°	90290-20N°	Z0000127108
90101-18N	90151-15	90185-16	90190-20N°	90194-18°	90284-24N	90290-22	Z0000127109
90101-18N°	90151-15N	90185-16°	90190-22	90194-18N	90284-24N°	90290-22°	Z0000130686
90101-20	90151-15N°	90185-16N	90190-22°	90194-18N°	90285-16	90290-22N	